


Care and Cleaning

To ensure optimal performance from your digitizer, it is important to clean several critical parts of the digitizer on a regular basis. There is a cleaning cloth come with your package for the purpose of using to clean the glass surface and diffuser inside your digitizer regularly. Please follow the steps below in this document to perform the cleaning; or, refer to the section “Care and Cleaning” in the User’s Manual for more details.


Open the Front Cover

Before you start cleaning each component, you have to open the front cover of the digitizer. To do do, push down two Release buttons at both sides of the front cover of the digitizer and open the front cover, as shown in the graphic right.

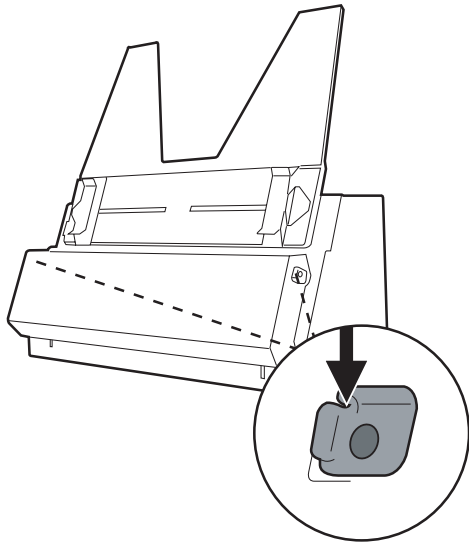

Cleaning the Glass Surface and the Diffuser

First, use the cleaning cloth come with your package or a soft and non-abrasive, lint-free cloth, clean the inner glass surface, located on the digitizer. Next, clean the white diffuser located at the middle of the front cover of the digitizer.

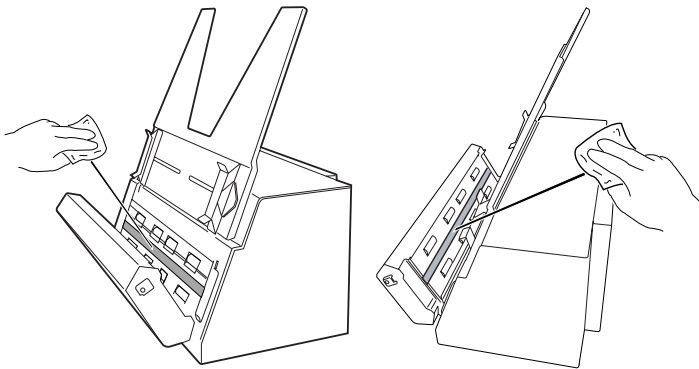


Clean the glass surface

Clean the diffuser

Close the Front Cover


After cleaning all components mentioned in paragraphs, please follow the steps below to close the front cover of the digitizer:

1. Push the cover forward slightly toward the digitizer.
2. With both hands, gently push two sides of the front cover back into the digitizer until you hear a “click”.

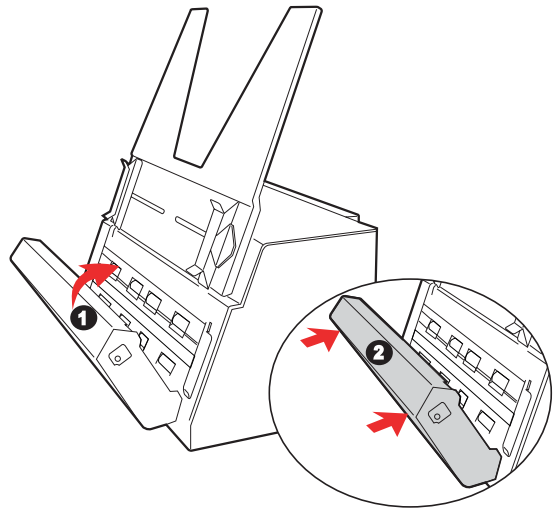

Cleaning the Rollers

Clean the rollers with a lint-free cloth or a swab moistened with alcohol or ECO cleaner, as shown in the graphic below. Remember to clean all the rollers located at the front cover and the digitizer.

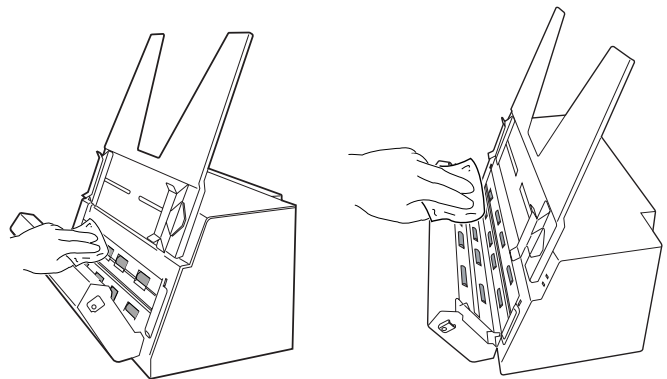

Rollers on the digitizer

Rollers at the front cover

NOTE: Do not use any detergents, synthetic cleaning solutions, cleaning naphtha, or other solvents to clean the glass surface or diffuser directly. To remove stains, use cotton swabs moistened with rubbing alcohol.